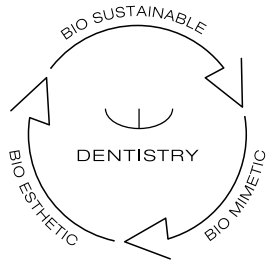
INDICATION GUIDE

DISCOVER YOUR SMILE

edelweiss
DENTISTRY

shaping the future of dentistry

AT ONE GLANCE...

Stephan Lamp
CEO & Founder of
edelweiss dentistry

Claudio Novelli
Clinical and
Scientific Director

edelweiss dentistry stands for the alliance between inspiration and technical know-how. VENEERs from edelweiss is state of the art for modern and minimally invasive esthetic dentistry. For the first time in the history of dental, it is now possible to work with prefabricated veneers made from nano-hybrid composite using modern laser technology. Never before has it been feasible to directly create the natural shape and youthful luminance of a tooth, so easily and perfectly in only one appointment. Its versatile area of application together with its time and cost saving procedure make edelweiss VENEERs a sound investment in the future with the best interest of the patient in mind. Convince yourself!

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DIRECT SYSTEM

THE PROGRESSIVE TECHNICAL CONCEPT

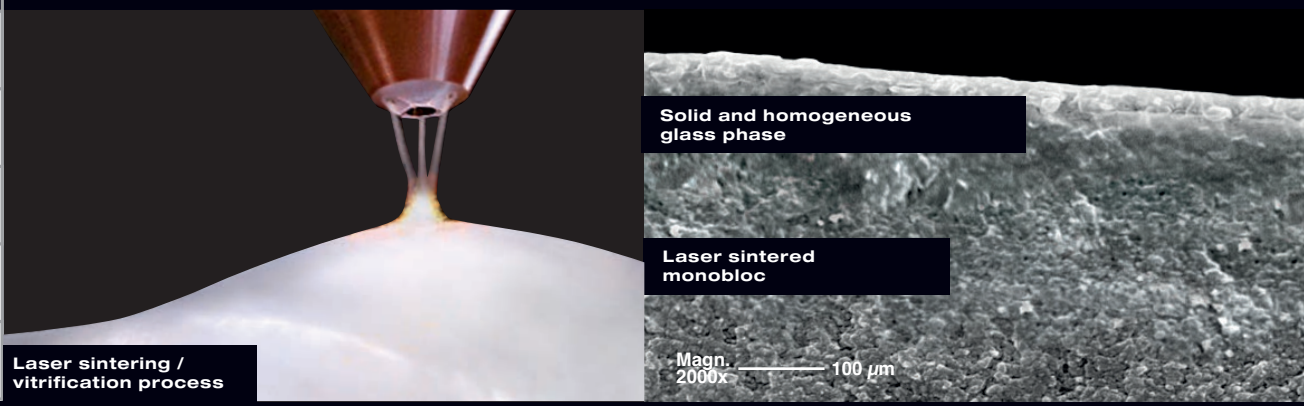
By combining and refining the latest composite & laser technology and achieving the optimum surface, edelweiss is the natural contemporary restorative option.

The difference is in it's similarity to nature.

edelweiss	VENEER
Flexural Strength	200 MPa
Compressive Strength	550 MPa
Flexural Modulus	20 GPa
Surface Hardness	100 HV
Filler Ratio	82 %

(Source: internal data from edelweiss dentistry)

SHADES

edelweiss SHADE SYSTEM

ENAMEL **VENEER & OCCLUSIONVD** + DENTIN / ENAMEL

edelweiss VENEERS & OCCLUSIONVD consist of the shade Enamel - Vita Enamel A0. The respective dentin and enamel shades used to cement the edelweiss VENEER & OCCLUSIONVD will determine the final shade tone of the restoration.

Example: edelweiss VENEER (Vita Enamel Shade A0) cemented with Dentin Shade A3 (Vita Dentin Shade A3) will result in the Vita Dentin Shade A3.

ENAMEL

Shade of edelweiss
VENEER & OCCLUSIONVD

Cervical area 0.2 mm

Facial area 0.6 mm

Incisal edge 1.0 - 1.3 mm

MAXILLA

MANDIBULA

Marginal area: 0.2 mm

Occlusion area: 0.6 - 1.3 mm

DIRECT SYSTEM

VENEER

OCCLUSIONVD

INDICATIONS

Anterior and posterior restorations
Tooth discolorations
Anatomical deformities
Diastema
Erosion & Attrition
Customized Veneers, Crowns & Bridges
Crowns & Crown facings
Lifting the vertical dimension (CMD)

ADVANTAGES

One appointment
Minimally invasive
Natural look
Long-lasting
Biomechanical
Biocompatible
Antibacterial
Economical

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MISALIGNED TEETH / OLD RESTORATIONS

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FR: Malformations anatomiques

IT: Deformità anatomiche

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ANATOMICAL DEFORMITIES / LENGTHENING OF THE INCISAL EDGE

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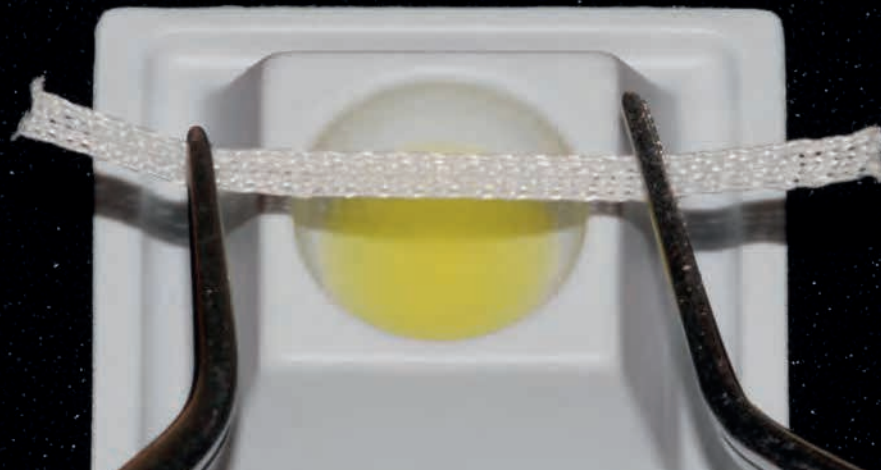
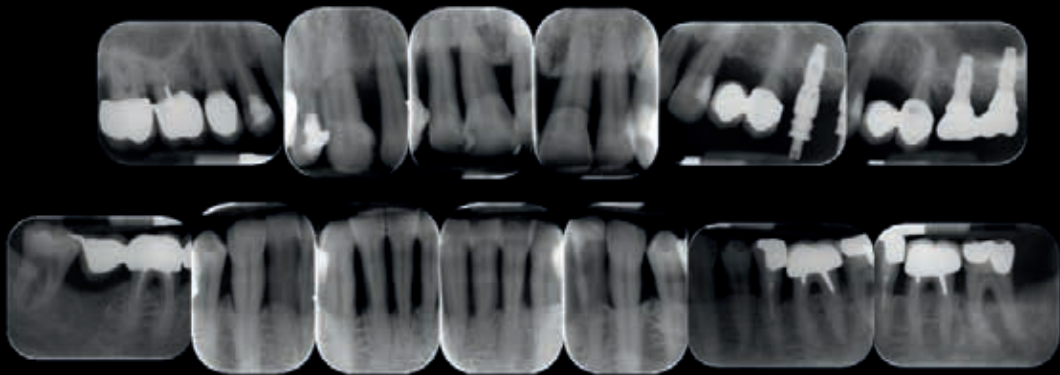
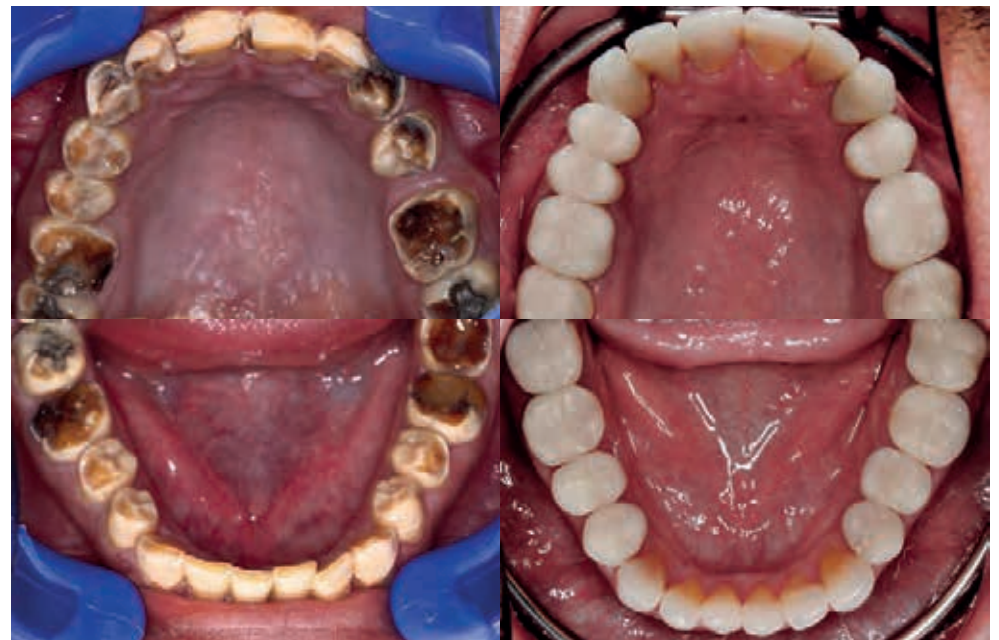
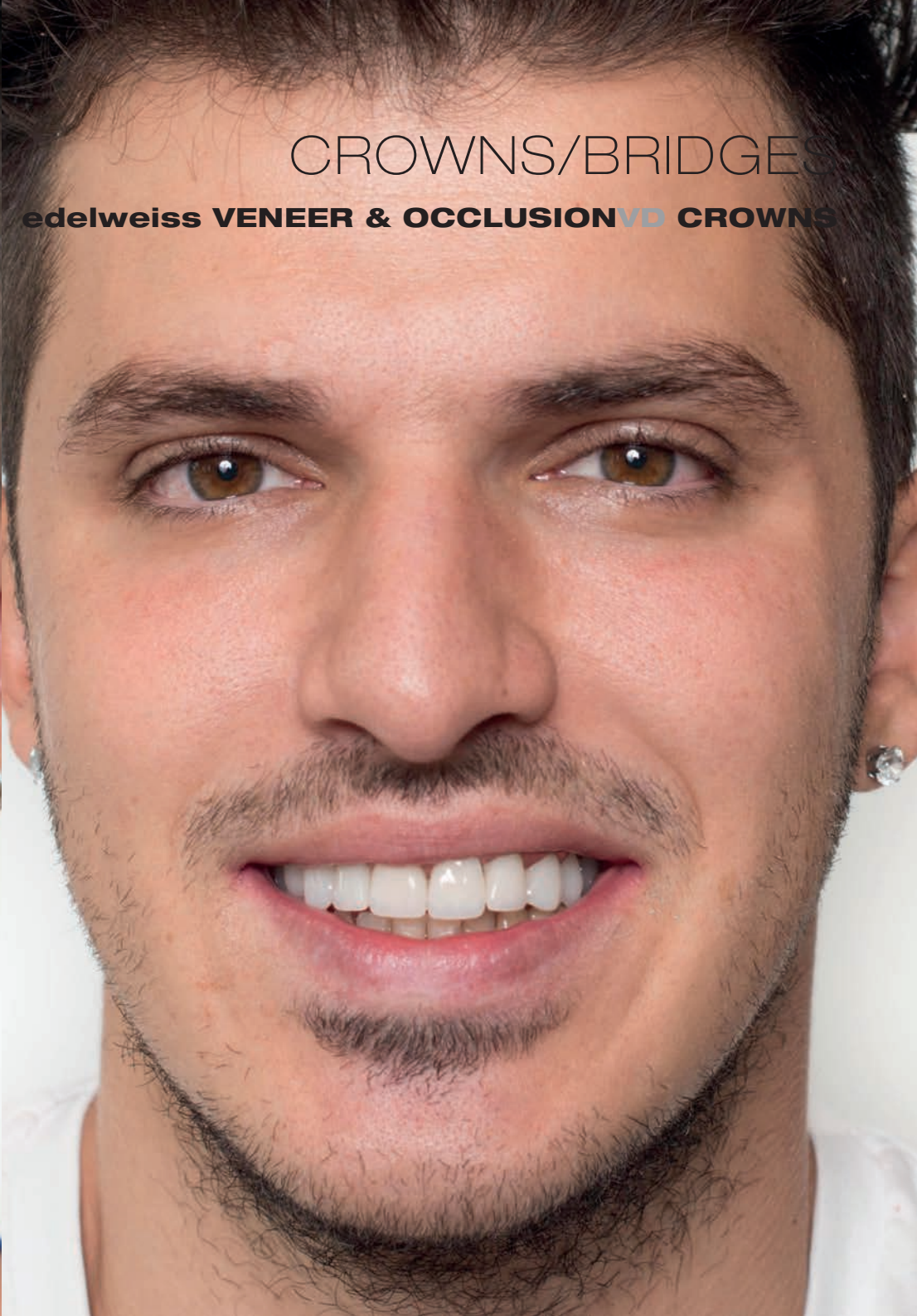
shaping the future of dentistry

edelveise dentistry ® and the edelveise Logo ® are registered trademarks of edelveise dentistry products gmbh • Austria

© 2022 edelveise dentistry products gmbh • Austria. All material contained in this brochure are the property of edelveise dentistry products gmbh • Austria. No part of this publication may be reproduced, distributed, or transmitted in any form or by any means, without the prior written permission of the edelveise dentistry products gmbh • Austria. All Rights Reserved.

edelveise dentistry products gmbh • Austria
office@edelveissdentistry.com
www.edelveissdentistry.com

816005 / 2022-04